

Programming Guide for Quantum

This article explains how to program both driving and seating modes in Ability Drive®. For the mode button to be turned on to access seating the Enhanced Display needs to be mounted next to the eye gaze device. Please contact us and we will remote into the eye gaze device and turn the mode button on. Phone (425)242-3902 or support@tolttechnologies.com

Quantum Programming Guide for Quantum QLogic 3.0

First, plug in the hand held programmer.

Step 1: Select Program Adjustments from the Main Menu

Step 2: Select Startup Configuration from the Program Adjustments Menu

Set Input Device = Input device selection screen

Set Input Device Selection = Power on Device

Step 3: Select Enhanced Display from the Program Adjustments Menu

Select Profile Setup

Set Profile 1 = Drive 1 (*Slow Indoor Driving*)

Set Profile 2 = Disabled (*Optional Step: Drive 2 - Outdoor Moderate Driving*)

Set Profile 3 = Disabled

Set Profile 4 = Disabled

Set Profile 5 = Disabled

Set Profile 6 = Seat

Select Input Configuration

Set ED Input Configuration = 4 Switch

Select Device Options / Timing

Set Device Double Command = Disabled

Select D-Sub (9-pin)

Set D-Sub Supervision = Disabled

Set Mode Long Command = Inactive

Set Mode Double Command = Inactive

Set D-Sub Mode E-Stop = Off

Select Switch Option / Timing

Set On / Off Jack Switch Type = 1 Switch

(this is the buddy button to turn off the chair)

Select Jack Command

Select Jack 1

Set On / Off Jack Short Command = Power Off

Select Menu Settings

Set Menu Navigation Mode = Manual

Select Standby Select

Set Standby Select = Disabled

Step 3: Select Enhanced Display from the Program Adjustments Menu (continued)

Select Sleep

Set Sleep Power Mode = Disabled

Select Latch Driving

Set Latch Driving Visibility = Disabled

Select User Speed Adjust

Set User Speed Adjust = Disabled

Select Profile Change Stop

Set Profile Stop Drive = Enabled

(this setting is important to ensure that the chair comes to a stop before accessing seating)

Step 4: Select Seat from the Program Adjustments Menu

Select Seat Settings

Set Seat Navigation = Standard

Set Seat Head Left / Right Swap = No

Set Seat Enabled While Driving = No